
Vis-Rx[®]

Micro-Imaging Catheter
High-Frequency OCT

English	4
Čeština	6
Српски	8
Dansk	10
Deutsch	12
Eesti.....	14
Ελληνικά.....	16
Español	18
Français	20
Hrvatski.....	22
Italiano	24
Latviešu.....	26
Lietuvių kalb	28
Nederlands.....	30
Norsk.....	32
Polski.....	34
Português.....	36
Slovenčina	38
Suomi	40
Svenska.....	42

Figure: Vis-Rx Micro-Imaging Catheter and Probe Interface Module (PIM) / Obrázek: Mikro-zobrazovací katétr Vis-Rx a Modul rozhraní sondy (PIM) / Слика: Vis-Rx kateter za mikro snimanje i modul interfejsa sonde (PIM) / Figur: Vis-Rx mikro-billeddannelseskatheter og probe-interfacemodul (PIM) / Abbildung: Vis-Rx Mikro-Bildgebungskatheter und Sondenschnittstellenmodul (PIM) / Joonis: Vis-Rx-i mikropildistuskateeter ja sondi liidesemoodul (PIM) / Εικόνα: Καθετήρας μικροαπεικόνισης Vis-Rx και Μονάδα διεπαφής αισθητήρα (PIM) / Figura: Catéter Vis-Rx para la obtención de microimágenes y módulo de contacto de la sonda (PIM) / Figure : Cathéter de micro-imagerie Vis-Rx et module d'interface de sonde (PIM) / Oblik: Vis-Rx kateter za mikro snimanje i modul sučelja sonde (PIM) / Figura: Catetere per micro-imaging Vis-Rx e modulo interfaccia-sonda (PIM) / Attēls: Vis-Rx mikroattēlveidošanas katetra un Zondes saskarnes modulis (PIM) / Paveikslas: „Vis-Rx Micro-Imaging“ kateterio ir zondo sąsajos modulis (PIM) / Afbeelding: Vis-Rx Micro-Beeldvormingskatheter en Probe Interface Module (PIM) / Figur: Vis-Rx Micro-Imaging Catheter og Probe Interface Module (PIM) / Rysunek: Cewnik mikroobrazowy Vis-Rx oraz moduł interfejsu sondy (PIM) / Figura: Cateter para aquisição de microimagens Vis-Rx e módulo de interface da sonda (PIM) / Obrázok: Katéter na mikrozobrazovanie Vis-Rx a modul rozhrania sondy (PIM) / Kuva: Vis-Rx-mikrokuvantamiskatetri ja koettimen liitäntämoduuli (PIM) / Figur: Vis-Rx-mikroavbildningskatetern och Probe Interface Module (PIM)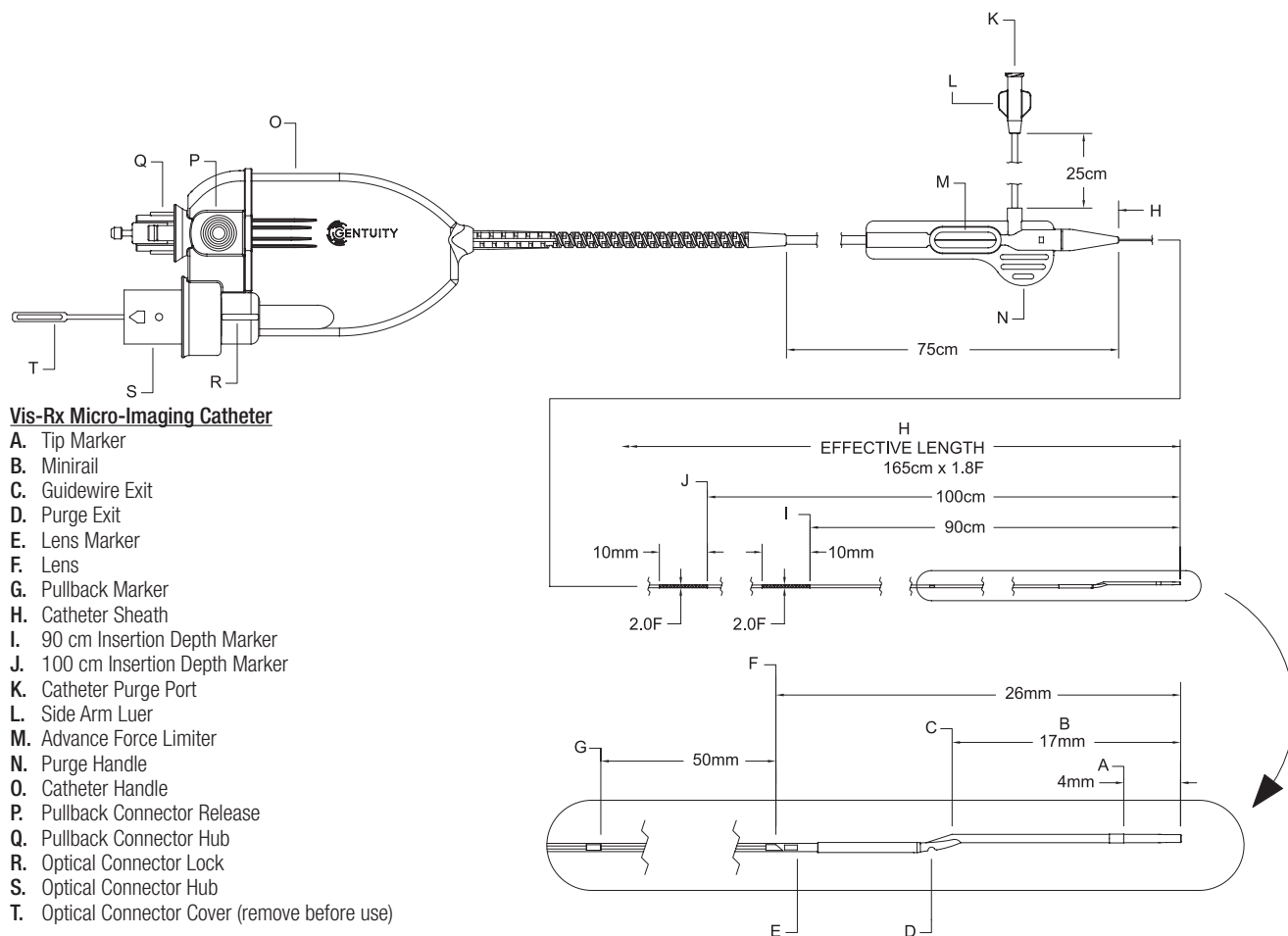


Vis-Rx Micro-Imaging Catheter

- A. Tip Marker
- B. Minirail
- C. Guidewire Exit
- D. Purge Exit
- E. Lens Marker
- F. Lens
- G. Pullback Marker
- H. Catheter Sheath
- I. 90 cm Insertion Depth Marker
- J. 100 cm Insertion Depth Marker
- K. Catheter Purge Port
- L. Side Arm Luer
- M. Advance Force Limiter
- N. Purge Handle
- O. Catheter Handle
- P. Pullback Connector Release
- Q. Pullback Connector Hub
- R. Optical Connector Lock
- S. Optical Connector Hub
- T. Optical Connector Cover (remove before use)

Mikro-zobrazovací katétr Vis-Rx / Vis-Rx kateter za mikro snimanje / Vis-Rx mikro-billeddannelseskatheter / Vis-Rx Mikro-Bildgebungskatheter / Vis-Rx-i mikropildistuskateeter / Καθετήρας μικροαπεικόνισης Vis-Rx / Catéter Vis-Rx para la obtención de microimágenes / Cathéter de micro-imagerie Vis-Rx / Vis-Rx kateter za mikro snimanje / Catetere per micro-imaging Vis-Rx / „Vis-Rx Micro-Imaging“ kateteris / Vis-Rx Micro-Beeldvormingskatheter / Vis-Rx Micro-Imaging Catheter / Cewnik mikroobrazowy Vis-Rx / Katéter na mikrozobrazovanie Vis-Rx / Cateter para aquisição de microimagens Vis-Rx / Vis-Rx-mikrokuvantamiskatetri / Vis-Rx-mikroavbildningskatetern

- A. Značka hrotu / Маркер са врхом / Spidsmarkør / Spitzenmarkierung / Otsa marker / Δείκτης σημείου τέλους / Marcador de punta / Repère d'extrémité / Marker vrha / Marcatore della punta / Uzgāja markieris / Žymeklis su antgaliu / Tipmarkering / Spissmarkør / Znacznik końcówek / Značka hrotu / Marcador da ponta / Kärjen merkki / Spetsmarkör
- B. Minikolejnice / Мини-шина / Miniskinne / Minirail / Minisiin / Τύπου «monorail» / Monorail / Monorail / Mini uvodnica / Minirail / Minisiede / Mini bégelis / Minirail / Miniskinne / Kořícówka minirail / Monorail / Minirail / Miniskisko / Miniskena
- C. Výstup vodičho drátu / Излаз водич-жиче / Guidewire udgang / Führungsdraht-Ausgang / Juhttraadi väljumine / Έξοδος αόρματος-οδηγού / Salida del cable guía / Évacuation du fil-guide / Izlaz žice vodilice / Uscita del filo guida / Virzoša vada izeja / Kreipiamosios vielos išėjimas / Geleidingsdraaduitgang / Ledetrådgang / Wyjście przewodnika / Výstup vodiaceho drôtu / Saída do fio-guia / Ohjainvaijerin ulostulo / Styrtrådsutgång
- D. Výstup proplachování / Излаз за чишћење / Skylningsudgang / Spülaustrag / Puhaustuse väljumine / Έξοδος καθαρισμού / Salida de purga / Évacuation de purge / Izlaz za čišćenje / Uscita di spurgo / Skalošanas izeja / Prapūsti išėjimą / Reinigingsuitgang / Renseutgang / Wyjście płukania / Plinaci výstup / Saída de purga / Huuhtelun ulostulo / Spolningsutgång
- E. Značka čočky / Маркер сочива / Linsemarkør / Linsemarkierung / Objektiiv marker / Δείκτης φακού / Marcado de lente / Repère de lentille / Marker leče / Marcatore della lente / Lēcas markieris / Lęšio žymeklis / Lensmarkering / Linsemarkør / Znacznik soczewki / Značka objektivu / Marcador da lente / Linssin merkki / Linsmarkör
- F. Čočka / Социво / Linse / Linse / Objektiiv / Φακός / Lente / Lentille / Leča / Lente / Lēca / Lęsis / Lens / Linse / Soczewka / Objektiiv / Lente / Linssi / Lins
- G. Značka stažení / Маркер за повлачење / Pullbackmarkør / Rückzugsmarkierung / Tagasitõmbemarker / Δείκτης Pullback / Marcador de retirada / Repère de pull-back / Marker povlačenja / Marcatore di pullback / Atpakaļgaitas markieris / Ištvausti žymeklj / Pullback-markering / Tilbaketrekkingsmarkør / Znacznik ruchu wstecznego / Značka pullbacku / Marcador de pullback / Takaisinvedon merkki / Pullback-markör
- H. Plášť katétru / Катетерски омотач / Kateterkappe / Katheterhülse / Kateetri ümbris / Περιβλήμα καθετήρα / Vaina del catéter / Gaine du cathéter / Ovojnica katetera / Guaina del catetere / Katetra aizsargapvalks / Kateterio apvalkalas / Katheterhuls / Kateterkappe / Koszulka cewnika / Plášť katétra / Bainha do cateter / Katetrin holkki / Kateterskida
- I. Značka hloubky zavedení 90 cm / Маркер дубине уметања од 90 cm / 90 cm indføringsdybde marker / 90 cm Einschubtiefenmarkierung / 90 cm sisestussügavuse marker / Δείκτης βάθους εισαγωγής 90cm / Marcador de profundidad de la inserción de 90 cm / Repère de profondeur d'insertion à 90 cm / Marker dubine umetanja od 90 cm / Marcatore di profondità di inserimento a 90 cm / 90 cm ievadišanas dziļuma markieris / 90 cm iļerpimo gyljo žymeklis / 90 cm Schachtmarkierung / Markør for 90 cm innsettingsdybde / Znacznik głębokości wprowadzania 90 cm / Značka hlčky zavedenia 90 cm / Marcador de profundidade de inserção aos 90 cm / 90 cm:n asetussyvyysmerkki / 90 cm-markör för insättningsdjup

- J. Značka hloubky zavedení 100 cm / Маркер глубины вметания од 100 см / 100 cm indförsringsdybemarkör / 100 cm Einschubtieftiefenmarkierung / 100 cm sisestussügavuse marker / Δείκτης βάθους εισαγωγής 100cm / Marcador de profundidad de la inserción de 100 cm / Repère de profondeur d'insertion à 100 cm / Marker dubine umetanja od 100 cm / Marcatore di profondità di inserimento a 100 cm / 100 cm ievadišanas dziļuma marķieris / 100 cm įterpimo gylis žymeklis / 100 cm Schachtmarkierung / Markör för 100 cm insettingsdybde / Znacznik głębokości wprowadzania 100 cm / Značka hĺbky zavedenia 100 cm / Marcador de profundidade de inserção aos 100 cm / 100 cm:n asetussyvyysmerkki / 100 cm-markör för insättningsdjup
- K. Port proplachování katétru / Прикључак за чишћење катетера / Kateterskylningsport / Katheterspülanschluss / Kateetri puhastusport / Θύρα καθαρισμού καθετήρα / Puerto de purga del catéter / Port de purge du cathéter / Otvor za čišćenje katetere / Porta di spurgo del catetere / Katetra skalošanas ports / Kateterio prapūtimo prievadas / Katheterreinigungsport / Kateterskylleport / Port plukania cewnika / Plniaci port katetra / Porta de purga do cateter / Katetrin huuhdeluportti / Spolningsport för kateter
- L. Postranní rameno Luer / Луер прикључак бочне руке / Luer til sidearm / Seitenarm-Luer / Külgharu Luer / Πλάγιός Βραχίονας Luer / Luer del brazo lateral / Bras latéral Luer / Priključak s bočnim otvorom / Luer del braccio laterale / Atzara Luera tipa savienojums / Šoninės svirtelės Luerio jungtis / Luer-zijarm / Sidearmluer / Złącze typu luer na ramieniu bocznym / Spojka typu Luer na bočnom ramene / Luer do braço lateral / Sivuvarren luer / Sidarmuer
- M. Pirmsunūtā omezovač spēli / Граничник силе напредовања / Advance Force Limiter / Vorschubkraftbegrenzer / Edasisurumise piiraja / Περιοριστής δύναμης προώθησης / Advance Force Limiter / Advance Force Limiter / Graničnik sile uvlačenja / Advance Force Limiter / Virzšanas uz priekšu spēka ierobežotājs / Postūmio jėgos ribotuvus / Advance Force Limiter / Advance Force Limiter / Ogranicznik siły wprowadzania / Obmedzovač sily posúvania / Limitador da força de avanço / Siirtymisvoiman rajoitin / Advance Force Limiter
- N. Proplachovací rukojet / Ручица за чишћење / Skylningshåndtag / Spülgriff / Puhastuskäepide / Λαβή καθαρισμού / Mango de purga / Poignée de purge / Ručica čišćenj / Impugnatura di spurgo / Skalošanas rokturis / Prapūtimo rankenėlė / Reinigingshandvat / Rensehåndtak / Uchwył do plukania / Rukováť plnenia / Pega de purga / Huuhdelun kahva / Spolningshandtag
- O. Rukojet katétru / Ручица катетера / Kateterhåndtag / Kathetergriff / Kateetri käepide / Λαβή καθετήρα / Mango del catéter / Poignée du cathéter / Ručica katetere / Impugnatura del catetere / Katetra rokturis / Kateterio rankenėlė / Katheterhandvat / Kateterhåndtak / Uchwył cewnika / Rukováť katétra / Pega do cateter / Katetrin kahva / Kateterhandtag
- P. Uvolnění konektoru stažení / Отпущање конектора за повлачење / Udløsning af pullback-tilslutning / Rückzugssteckanschlussfreigabe / Tagasitömbekonnektori vabastus / Απελευθέρωση συνδέσμου του Pullback / Liberación del conector de retirada / Libération du connecteur de pull-back / Otpuštanje konektora povlačenja / Rilascio del connettore di pullback / Atpakalgaitas savienotāja atbrīvošanas mehānisms / Ištraukimo jungties atleidimas / Pullback Connector Release / Frigjøring av tilbaketrekkingskobling / Zwolnienie złącza ruchu wstecznego / Uvolnienie konektora na pullback / Desengate do conector de pullback / Takaisinvedon liittimen vapautus / Frigöring av pullback-anlutning
- Q. Rozbočovač konektoru stažení / Чвориште конектора за повлачење / Samling for pullback-tilslutning / Rückzugssteckanschluss / Tagasitömbekonnektori jaotur / Μπλοκ συνδέσμου του Pullback / Elevación del conector de retirada / Bloc connecteur de pull-back / Čvorište konektora povlačenja / Hub del connettore di pullback / Atpakalgaitas savienojuma mezgls / Ištraukimo jungties šakotuvus / Pullback Connector Hub / Nav for tilbaketrekkingskobling / Koncentrator złącza ruchu wstecznego / Rozbočovač konektora na pullback / Encaixe do conector de pullback / Takaisinvedon liittimen napa / Pullback-anlutningsnav
- R. Zámek optického konektoru / Закључавање оптичког конектора / Optisk tilslutningslås / Optikanschlussschloss / Optilise konnektori lukk / Κλειδώμα οπτικού συνδέσμου / Bloqueo del conector óptico / Verrou du connecteur optique / Sustav zaključavanja optičkog konektora / Blocco del connettore ottico / Optiskā savienotāja fiksators / Optinės jungties užraktas / Optical Connector Lock / Lås for optisk tilkobling / Blokada złącza optycznego / Zámok optického konektora / Bloqueo do conector ótico / Optisen liittimen lukitus / Optiskt anslutningslås
- S. Rozbočovač optického konektoru / Чвориште оптичког конектора / Samling for optisk tilslutning / Optikanschlusssnabe / Optilise konnektori jaotur / Μπλοκ οπτικού συνδέσμου / Elevación del conector óptico / Bloc connecteur optique / Čvorište optičkog konektora / Hub del connettore ottico / Optiskā savienotāja mezgls / Optinės jungties šakotuvus / Optical Connector Hub / Nav for optisk tilkobling / Koncentrator złącza optycznego / Rozbočovač optického konektora / Encaixe do conector ótico / Optisen liittimen napa / Optiskt anslutningsnav
- T. Kryt optického konektoru (před použitím sejměte) / Поклопач оптичког конектора (уклоните пре употребе) / Optisk tilslutningsdæksel (fjernes for brug) / Abdeckung des Optikanschlusses (vor Gebrauch entfernen) / Optilise konnektori kaas (eemaldada enne kasutamist) / Καλύπτρα οπτικού συνδέσμου (να αφαιρεθεί πριν από τη χρήση) / Cubierta del conector óptico (quitar antes de usar) / Capuchon du connecteur optique (à retirer avant utilisation) / Poklopac optičkog konektora (ukloniti prije upotrebe) / Tappo del connettore ottico (rimuovere prima dell'uso) / Optiskā savienotāja vāks (pirms lietošanas noņemt) / Optinės jungties dangtelis (prieš naudojimą nuimti) / Optical Connector Cover (beschermhoes voor gebruik verwijderen) / Dæksel for optisk tilkobling (fjern for bruk) / Oslona złącza optycznego (zdyjąć przed użyciem) / Kryt optického konektora (pred použitím ho zložte) / Tampa do conector ótico (retirar antes de utilizar) / Optisen liittimen suoja (poista ennen käyttöä) / Överdrag för optisk anslutning (ta bort före användning)

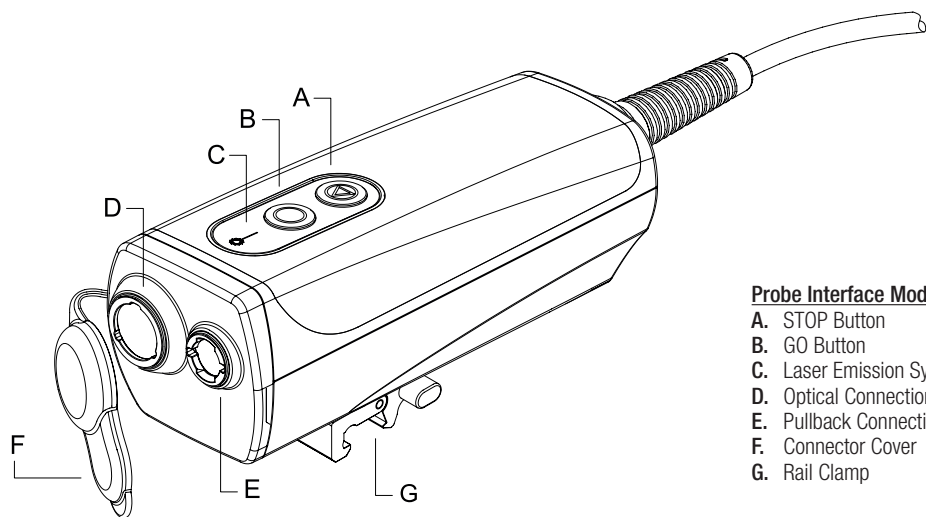Probe Interface Module (PIM)

- A. STOP Button
- B. GO Button
- C. Laser Emission Symbol
- D. Optical Connection Port
- E. Pullback Connection Port
- F. Connector Cover
- G. Rail Clamp

Modul rozhraní sondy (PIM) / Модул интерфејса сонде (PIM) / Probe-interfacemodul (PIM) / Sondenschnittstellenmodul (PIM) / Sonodi liidesemoodul (PIM) / Μονάδα διεπαφής αισθητήρα (PIM) / Módulo de contacto de la sonda (PIM) / module d'interface de sonde (PIM) / Modul sučelja sonde (PIM) / Modulo interfaccia-sonda (PIM) / Zondes saskarnes modulis (PIM) / Zondo sąsajos modulis (PIM) / Probe Interface Module (PIM) / Probe Interface Module (PIM) / Modul interfejsu sondy (PIM) / Modul rozhrania sondy (PIM) / Módulo de interface da sonda (PIM) / Koettimen liitântämoduuli (PIM) / Probe Interface Module (PIM)

- A. Tlačítko STOP / Дугме за ЗАВСТАВЉАЊЕ / STOP knap / STOP-Taste / Seiskamisnupp / Κομπι STOP / Botón STOP / Bouton STOP / Dugme ZAUSTAVI / Pulsante STOP / Poga STOP / STOP mygtukas / STOP-knop / STOP-knapp / Przycisk STOP / Tlačítko STOP / Botão STOP / STOP (PYSÄYTÄ) -painike / STOP-knapp
- B. Tlačítko GO / Дугме за ПОКРЕТАЊЕ / GO knap / GO-Taste / Käivitusnupp / Κομπι GO / Botón GO / Bouton GO / Dugme POČNI / Pulsante GO / Poga GO / GO mygtukas / GO-knop / GO-knapp / Przycisk GO (ROZPOCZNIJ) / Tlačítko GO / Botão GO / GO (SIIRRY) -painike / GO-knapp
- C. Symbol emise laseru / Символ ласерске емисије / Laseremissionssymbol / Laseremissionssymbol / Laserikiirguse sümbol / Σύμβολο εκπομπής λέιζερ / Símbolo de emisión láser / Symbole d'émission laser / Simbol laserske emisije / Simbolo di emissioni laser / Lázera starojuma simbols / Lazerio emisijos simbolis / Laseremissie Symbol / Symbol for laserutslipp / Symbol emisji lasera / Symbol laserového žiarenia / Símbolo de emissão do laser / Lasersäteilyn symboli / Laseremissionssymbol
- D. Optický port pro připojení / Прикључак за оптичку везу / Optisk tilslutningsport / Optik-Steckanschluss / Optilise konnektori port / Θύρα οπτικής σύνδεσης / Puerto de conexión óptica / Port de connexion optique / Priključak optičke veze / Porta di connessione ottica / Optiskā savienojuma ports / Optinės jungties prievadas / Optische Aansluiting Poort / Port for optisk tilkobling / Port złącza optycznego / Port optického konektoru / Porta de ligação ótica / Optikan liitântäportti / Optisk anslutningsport
- E. Port spojení stažení / Прикључак за везу са повлачењем / Port til pullback-tilslutning / Rückzugssteckanschluss / Tagasitömbekonnektori port / Θύρα σύνδεσης pullback / Puerto de conexión de retirada / Port de connexion de pull-back / Priključak povlačenja / Porta di connessione di pullback / Atpakalgaitas savienojuma ports / Ištraukimo jungties prievadas / Pullback Connector Port / Tilkoblingsport for tilbaketrekking / Port złącza ruchu wstecznego / Port pripojenia na pullback / Porta de ligação de pullback / Takaisinvedon liitântäportti / Pullback-anlutningsport
- F. Kryt konektoru / Поклопач конектора / Tilslutningsdæksel / Anschlussabdeckung / Cubierta del conector / Capuchon du connecteur / Poklopac priključka / Tappo dei connettori / Savienotāja vāks / Jungties dangtelis / Connectorhoes / Tilkoblingsdæksel / Oslona złącza / Kryt konektora / Tampa do conector / Liittimen suoja / Anslutningsöverdrag
- G. Kolejnicová svorka / Гезаљка шине / Skinneklemme / Schienenklemme / Piirdekinnituskamber / Συγκριτής προσάρτησης στη ράγα / Abrazadera del rail / Pince de fixation au rail / Držac uvodnice / Morsetto per binario / Sliedes skava / Bégio spaustukas / Railklem / Skinneklemme / Zacisk szynowy / Svorka na kolajničku / Grampo de corrimão / Kaiteen puristin / Skenklämma

Intended Use

The Genuity® HF-OCT Imaging System with Vis-Rx® Micro-Imaging Catheter is intended for intravascular imaging and is indicated for use in coronary arteries in patients who are candidates for transluminal interventional procedures. The Vis-Rx Micro-Imaging Catheter is intended for use in vessels 1.3 to 6.0 mm in diameter. The Vis-Rx Micro-Imaging Catheter is not intended for use in a target vessel that has undergone a previous bypass procedure.

Contraindications

Contraindications for use of the Genuity High-Frequency OCT Imaging System include:

- Bacteremia or sepsis
- Major coagulation system abnormalities
- Coronary artery spasm
- Severe hemodynamic instability or shock
- Total occlusion
- Large thrombus
- Acute renal failure
- Patients disqualified for CABG surgery
- Patients disqualified for PTCA

WARNINGS

- ⚠ Before using the Vis-Rx Micro-Imaging Catheter, review the *Genuity High-Frequency OCT Imaging System User Manual* for additional warnings and cautions.
- ⚠ The Genuity High-Frequency OCT Imaging System is intended for use only by medical personnel trained in its operation and skilled in the clinical procedures to be used.
- ⚠ Appropriate anticoagulant and vasodilator therapy must be used during the procedure as needed.
- ⚠ Observe all advancement and movement of the Vis-Rx Micro-Imaging Catheter under fluoroscopy. Always advance and withdraw the catheter slowly. Failure to observe device movement fluoroscopically may result in vessel injury or device damage.
- ⚠ Leave the guidewire engaged with the catheter at all times during use. Do not withdraw or advance the guidewire prior to withdrawing the catheter.
- ⚠ If resistance is encountered during the advancement or withdrawal of the Vis-Rx Micro-Imaging Catheter, stop manipulation and evaluate under fluoroscopy. If the cause of resistance cannot be determined or mitigated, carefully remove the catheter and guidewire together.
- ⚠ The catheter should never be forced into lumens that are narrower than the catheter body.
- ⚠ When advancing or withdrawing a catheter with a minirail tip through a stented vessel, the catheter may engage the stent between the junction of the catheter and guidewire, resulting in entrapment of catheter/guidewire, catheter tip separation, and/or stent dislocation.
- ⚠ The catheter is sterilized by irradiation and is intended for one time use only. Do not reuse, re-sterilize, or reprocess. Reuse or re-processing could result in a degradation of catheter material or patient infection.
- ⚠ Non-pyrogenic. Do not use if the package is opened or damaged.
- ⚠ Do not use the catheter after the expiration date, or if there is not a date on the package.
- ⚠ The catheter is not compatible with magnetic resonance imaging (MRI).

CAUTIONS

- ⚠ Prior to use and for more detailed information, please review the *Genuity High-Frequency OCT Imaging System User Manual*.
- ⚠ Refer to the contrast media's instructions for general warnings and precautions relating to contrast media.

Complications

The risks involved in vascular imaging include those associated with all catheterization procedures. The following complications (listed alphabetically) may occur as a consequence of intravascular imaging and may

necessitate additional medical treatment including surgical intervention.

- Acute myocardial infarction or unstable angina
- Allergic reaction to contrast media
- Arterial dissection, injury, or perforation
- Cardiac arrhythmias
- Coronary artery spasm
- Death
- Embolism
- Myocardial ischemia
- Renal insufficiency from contrast usage
- Thrombus formation

Intended Users

The device is intended to be used by physicians and technicians trained in the performance of catheter-based intracoronary interventional procedures. These physicians are interventional cardiologists. The interventional cardiologist will frequently be assisted by an interventional cardiology fellow, catheterization laboratory technician, or nurse.

Description

The Vis-Rx Micro-Imaging Catheter is a rapid exchange (RX) catheter with a 17 mm minirail tip designed for compatibility with a 0.014" (0.356 mm) guidewire. The catheter is 1.8 Fr in size, except at the insertion depth markers, where it is 2.0 Fr. The effective length of the catheter is 165 cm. The distal 30 cm has a hydrophilic coating applied to improve lubricity.

The Vis-Rx Micro-Imaging Catheter consists of two assemblies: an external catheter sheath and an imaging core (housing an optical fiber and lens assembly).

Proximal to the minirail tip is the imaging area. During image acquisition, the lens rotates within the imaging core to obtain a 360° image of the surface layer of the artery wall. The imaging core is automatically retracted within the external catheter sheath to obtain a continuous pullback image of the arterial segment.

Markers

There are three radiopaque markers on the catheter:

1. The most distal marker, the **tip marker**, is located 4 mm proximal to the tip of the catheter and is affixed to the catheter sheath.
2. The **lens marker** is located 5 mm distal to the lens on the imaging core.
3. There is an additional **pullback marker** placed 50 mm proximal to the lens on the imaging core.

Together, these markers enable confirmation of the catheter distal end, lens location, and imaging region. The lens and pullback markers move with the pullback, while the tip marker remains stationary.

The catheter sheath has two insertion depth markers at 90 cm and 100 cm from the distal tip that provide an indication of insertion depth.

Purging

A luer fitting on the side-arm at the proximal end of the catheter facilitates purging the central catheter lumen of the Vis-Rx Micro-Imaging Catheter with heparinized saline prior to use. A 3 ml syringe is provided to perform the catheter purge. The purge must be performed prior to insertion and imaging. The syringe should be left attached to the side-arm to allow repeated purging throughout the imaging procedure and to maintain a static pressure to prevent backflow.

Probe Interface Module (PIM)

The PIM provides both automated rotation and longitudinal pullback of the imaging core within the Vis-Rx Micro-Imaging Catheter. The PIM is rail-mountable and can be placed outside of the sterile field so that a non-sterile technician can connect the catheter to the PIM and operate it during the procedure.

The Vis-Rx Micro-Imaging Catheter connects to the Genuity High-Frequency OCT Imaging System through the PIM. All imaging core rotation and translational pullback is driven by the PIM and occurs inside the catheter sheath. For more

details on the PIM, see the *Genuity High-Frequency OCT Imaging System User Manual*.

USING THE CATHETER

The system provides illustrated instructions for connecting the catheter to the PIM. The instructions are displayed when you select the **Acquire** button if the catheter is not yet connected to the PIM.

Connecting the Catheter to the PIM

Note: **SO** indicates Sterile Operator. **NSO** indicates Non-Sterile Operator.

1. **NSO:** Attach the PIM to the surgical bed rail by hooking the top of the clamp to the rail and tilting downward.
2. **NSO:** Carefully open the catheter and syringe pouches and transfer the contents into the sterile field using sterile techniques.
3. **SO:** Using sterile techniques, pass the catheter connector to the NSO near the PIM.
4. **NSO:** Remove the cover from the PIM connector and the cover from the catheter optical connector.
5. **NSO:** Align the catheter connectors to the connection ports on the PIM and insert the catheter into the PIM until it snaps into place.
6. **NSO:** Rotate the optical connector lock clockwise to the **LOCKED** position.

When properly connected, the system progresses to the next stage of readiness.

CAUTIONS

- ⚠ Protect the PIM connection ports and the catheter connectors from fluids.
- ⚠ Do not touch the internal optics of the optical connector on the catheter or on the PIM.
- ⚠ Observe sterile technique when connecting the catheter to the PIM, which is outside of the sterile field.

Preparing the Catheter

1. **SO:** When ready to perform imaging, activate the hydrophilic coating by injecting heparinized saline into the hoop or wiping the distal segment with gauze moistened with heparinized saline.
2. **SO:** Fill the provided 3 ml syringe with heparinized saline and attach to the catheter purge port. Purge the lumen of the catheter with saline until 3-5 drops exit the distal purge exit.
3. **SO:** Leave the purge syringe connected so that the catheter lumen can be repurged if necessary.

CAUTION

- ⚠ Do not remove the syringe from the catheter purge port to prevent air from entering the purge lumen and to allow repurging as necessary.

Inserting and Positioning the Vis-Rx Micro-Imaging Catheter

1. **SO:** Insert the guidewire into the guidewire lumen of the Vis-Rx Micro-Imaging Catheter and advance the imaging catheter over the guidewire. Use the insertion depth gauges at 90 cm and 100 cm as guides.

CAUTION

- ⚠ Use a guidewire with a maximum outer diameter of 0.014" (0.36 mm) and a guide catheter with a minimum inner diameter of 0.068" (1.73 mm).
- 2. **SO:** Using fluoroscopy, position the imaging catheter in the region of interest, using the radiopaque markers at the tip, imaging lens, and 50 mm marker proximal to the lens for guidance.

WARNINGS

- ⚠ If resistance is encountered during advancement or withdrawal of the catheter, stop manipulation and evaluate under fluoroscopy. If the cause of resistance cannot be determined or mitigated, carefully remove the catheter and guidewire together.

- ⚠ Observe all advancement and movement of the imaging catheter in the vasculature under fluoroscopy.
- ⚠ If the imaging catheter becomes kinked, stop manipulating to avoid vessel injury or imaging catheter damage.
- ⚠ Leave the guidewire engaged with the catheter at all times during use.
- ⚠ To avoid blood vessel damage, maintain the position of the guidewire when manipulating the imaging catheter in the vessel.

CAUTIONS

- ⚠ To avoid catheter damage, make sure the PIM motor is NOT running when inserting the Vis-Rx Micro-Imaging Catheter into the guide catheter.
- ⚠ Always advance and withdraw the imaging catheter slowly.
- ⚠ To ensure imaging of the selected anatomy, do not move the guidewire after the imaging catheter is in position.

Confirming Imaging Settings

1. **NSO:** When ready, select **Acquire** on the touchscreen. If the catheter has not yet been connected, the system provides guidance.
2. **NSO:** Specify the following in the **Imaging Settings** window:
 - Select the **Pullback Settings**.
 - Confirm the default **Flush Media** type or select another **Flush Media** type.
 - In the **Automatic Flush Detection** section, select **ON** for automatic detection, or **OFF** for manual detection.

Notes:

- **Automatic Flush Detection (ON)** is the setting in which the system triggers a pullback automatically when a brief sequence of clearing is detected as a result of contrast injection. If clearing is not detected within 15 seconds after being enabled, the system displays a **Timeout** message. You can initiate pullback manually when in Automatic mode by selecting **Acquire** on the touchscreen or the **GO** button on the PIM.
- **Automatic Flush Detection (OFF)** is the manual setting. After clearing is observed, you select the **Acquire** button or press **Go** on the PIM to start the pullback. If you do not initiate pullback within 15 second after the system is enabled, the system displays a **Timeout** message.

WARNING

- ⚠ Selecting the wrong flush medium can cause measurement errors which could lead to incorrect treatment. Prior to acquisition, make sure the flush medium identified in the **Pullback Settings** window matches the flush medium you are using.
- 3. **NSO:** When ready, select **Confirm Settings** on the touchscreen.

Preparing the Flush Media

CAUTIONS

- ⚠ Refer to the contrast media's instructions for general warnings and precautions related to contrast media.
- ⚠ Make sure to use the contrast media and injectors according to specifications given by the manufacturers.
- SO:** Depending on the type of contrast media injection, do one of the following:
 - If using an **automated injector**, ensure the selected flush media is loaded in the injector and set the flush rate to 4 ml/sec or less, with a total volume of 16 ml or less and pressure limit of 300 psi (2068 kPa).
 - If using a **manual injection**, prepare a coronary control syringe capable of injecting up to 4 ml/sec for 3 to 4 seconds.

WARNING

- ⚠ Excessive flush rate and pressure may damage the blood vessel or devices used with the Vis-Rx Micro-Imaging Catheter. Low flush rate may result in a faint image.

Beginning Preview (Optional)

Perform the optional steps in Preview to confirm the position of the catheter within the vessel.

1. **NSO:** When the catheter is in position, select **Preview**. The PIM will activate and the OCT image is displayed. A **Calibration** window is also displayed for approximately 5 seconds.
2. **SO:** If necessary, use the 3 ml purge syringe to eliminate any blood from the catheter lumen.

Delivering 5 ml Flush Media to Confirm Alignment

Optionally, you can deliver flush media to confirm alignment of the guide catheter with the vessel.

1. **SO:** Deliver approximately 5 ml of flush media via the Y-connector to fill the guide catheter with flush media and to ensure proper guide catheter alignment with the ostium under fluoroscopic guidance.
2. Proceed to **Enable** step, or press **STOP** on the touchscreen (or on the PIM) to exit **Preview** mode.

CAUTIONS

- ⚠ To help ensure successful imaging, the guide catheter should be oriented to preferentially direct the flush media flow to the target artery.
- ⚠ To help ensure successful imaging, do not use a guide catheter with side holes.

Enabling the Start of High-Speed Rotation

NSO: With confirmation of proper guide position and imaging catheter position, select **Enable** on the touchscreen or press the green **Go** button on the PIM to start high-speed PIM rotation.

When the PIM reaches full speed and the catheter is calibrated, the system is ready for pullback. The 15-second clock begins, and the system prompts for injection of the flush media.

Note: If you do not inject the flush media and initiate pullback within the 15 seconds, the PIM rotation stops completely and a **Timeout** message is displayed. Select **OK** and then select **Enable** to enable the system when ready to image.

Injecting Flush Media and Initiating Pullback

WARNING

⚠ Before injecting flush media, be sure that the hemostasis valve is tightened to reduce the risk of unintended catheter movement or leaking of flush media during injection.

1. **SO:** Inject the flush media into the target artery via the Y-connector on the guide catheter.
2. **NSO:** Depending on the selected **Automatic Flush Detection** setting, do one of the following:
 - **Automatic Flush Detection (ON):** The system automatically initiates pullback when clearing is detected. Alternatively, you can manually initiate pullback prior to the automatic trigger by selecting **Acquire** on the touchscreen or by pressing the green **Go** button on the PIM.
 - **Automatic Flush Detection (OFF):** Select **Acquire** on the touchscreen or press the green **Go** button on the PIM to initiate the pullback and start capturing images once the blood is removed and a clear image displays (1-2 seconds after injecting the flush media). The image displays after completion of the pullback. The optical imaging core returns to the distal position within the sheath for further imaging.

CAUTION

⚠ If the optical imaging core encounters resistance while returning to the distal position within the sheath (due to a kink or blockage), the Advance Force Limiter will buckle to absorb the forward motion. If this occurs, carefully remove the imaging catheter from the guide catheter. Replace with a new catheter if further imaging is required.

Notes:

- **SO:** Do not remove the catheter until you confirm the image is acceptable.
- **NSO:** Do not disconnect the catheter from the PIM until you are completely finished with the system.

REMOVING THE CATHETER

After confirming that the image or images are acceptable, perform the following steps to remove the catheter:

1. **SO:** Under fluoroscopy, carefully remove the catheter from the guide catheter, checking the condition of the catheter, guidewire, and guide catheter during withdrawal.

WARNING

⚠ To avoid vessel damage or catheter damage, do not push the guide catheter further into the blood vessel when removing the catheter.

2. **SO:** After removing the catheter from the patient, use the 3 ml purge syringe to purge the Vis-Rx Micro-Imaging Catheter until 3-5 drops of saline exit the purge exit. Keep the purge syringe connected throughout the procedure.
3. **SO:** Wipe the distal section of the catheter with saline to remove external blood and flush media.

WARNING

⚠ Do not disconnect the Vis-Rx Micro-Imaging Catheter from the PIM until the procedure is complete to avoid a potential sterility breach.

When the catheter is removed from the guide catheter and no further imaging is required, it may be disconnected from the PIM.

DISCONNECTING THE CATHETER FROM THE PIM

1. **NSO:** Hold the catheter connector and unlock the optical connection by turning it counter-clockwise to the **UNLOCKED** position.
2. **NSO:** Grasp the pullback connector (by squeezing both sides) to remove the catheter from the PIM.
3. **NSO:** Place the protective port cover on the PIM to prevent damage to the catheter and pullback connection ports on the PIM.

CAUTIONS

- ⚠ When there is no catheter connected to the PIM, be sure the PIM connector cover is securely in place to protect from dirt and damage.
- ⚠ When not in use, always store the PIM in the PIM cradle with the PIM connector cover facing down.
- ⚠ Do not touch the internal optics of the optical connector on the catheter or on the PIM.
- ⚠ Protect the PIM connection ports and catheter connectors from fluids.
- 4. **NSO:** Dispose of the catheter in accordance with accepted medical practice and applicable laws and regulations.

WARNING

⚠ The catheter is intended for one time use only. Do not reuse, re-sterilize, or reprocess. Reuse or re-processing could result in a degradation of catheter material or patient infection.

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Notice - Information provided within this document is subject to change without notice. Gentuity, LLC shall not be liable for errors contained herein, or any omissions, or for incremental or consequential damages in connection with the furnishing, performance or use of this material.

Proprietary Information - This document contains proprietary information, which is protected by copyright.

Limited Warranty - The "Limited Warranty" shipped with Gentuity, LLC products serves as the sole and exclusive warranty provided by Gentuity, LLC with respect to the products contained herein.

Copyright - Copyright © 2023 Gentuity, LLC. All Rights Reserved.

Trademark Acknowledgments - Product names mentioned in this manual may be trademarks of their respective owners.

Gentuity - Telephone: 888-967-7628 - FAX: 978-319-4632 - General Inquiries: info@gentuity.com - Support and Service: support@gentuity.com - Website: www.gentuity.com

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established. The Summary of Safety and Clinical Performance (SSCP) is available in the European Database on Medical Devices (EUDAMED) where it is linked to the Basic UDI-DI for the Vis-Rx Micro-Imaging Catheter: 0859910007G30SN. The URL to access EUDAMED is: <https://ec.europa.eu/tools/eudamed>

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Copyright © 2023 Gentuity, LLC. Všechna práva vyhrazena.
Copyright – Copyright © 2023 Gentuity, LLC. Alle rettigheder forbeholdes.
Copyright © 2023 Gentuity LLC. Alle Rechte vorbehalten.
Derechos de autor - Copyright © 2023 Gentuity, LLC. Todos los derechos reservados.
Copyright - Copyright © 2023 Gentuity, LLC. Tous droits réservés.
Copyright - Copyright © 2023 Gentuity, LLC. Tutti i diritti riservati.